

#1661


# Translating cancer tissue methylation to cell-free DNA methylation for minimally invasive cancer detection.

E. Post, K. Roohollahi, D.C.L. Vessies, J. van den Berg, A. Foster, S. Patel, A. Vandor, D.J.S. Makarawung, F. van Asch  
RenovaroCube, Amsterdam, The Netherlands



## Introduction

- Cell-free circulating tumour DNA (ctDNA) is a minimally invasive biomarker
- There are large databases of cancer tissue research available
- Can we leverage tissue derived biomarkers to detect cancer in cfDNA?
- RenovaroCube ("The Cube") is an AI platform for cancer diagnostics
- TCGA is a large database containing cancer methylation data for multiple cancer types<sup>[1]</sup>
- cfMEDIP captures methylated cfDNA and allows for quantifying methylation levels in cfDNA

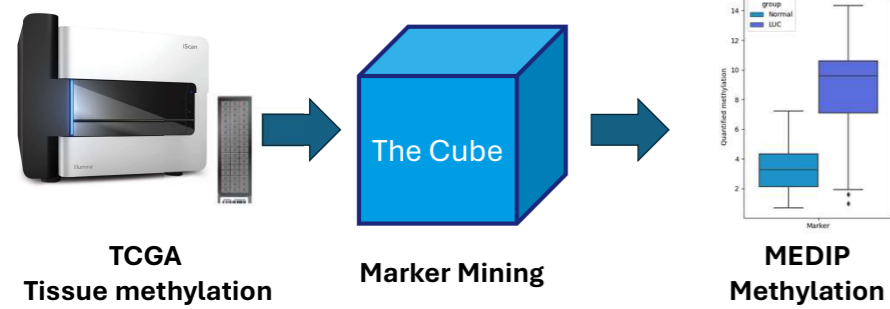

## Methods

- Marker mining
  - TCGA methylation array data
  - RenovaroCube biomarker mining
- Publicly available cfMEDIP data<sup>[2,3]</sup>
  - Convert Illumina biomarkers to cfMEDIP biomarkers
  - Perform differentially methylated region analysis (DMR)
- Training set:
  - Healthy donors (n=24)
  - Cancer patients (n=73)
    - 23 Colorectal cancer
    - 25 Lung cancer
    - 25 Breast cancer
- Test set:
  - Healthy donors (n=62)
  - Lung cancer patients (n=55)

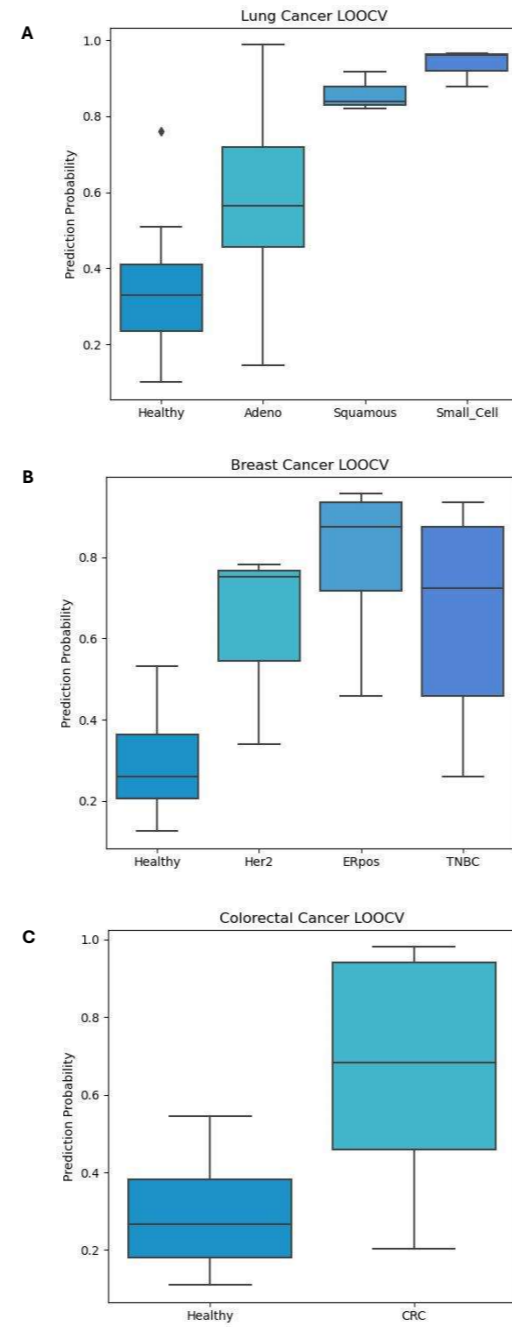


Figure 2. LOOCV Model prediction results for training sets. A: Lung cancer. B: Breast Cancer. C: Colorectal cancer

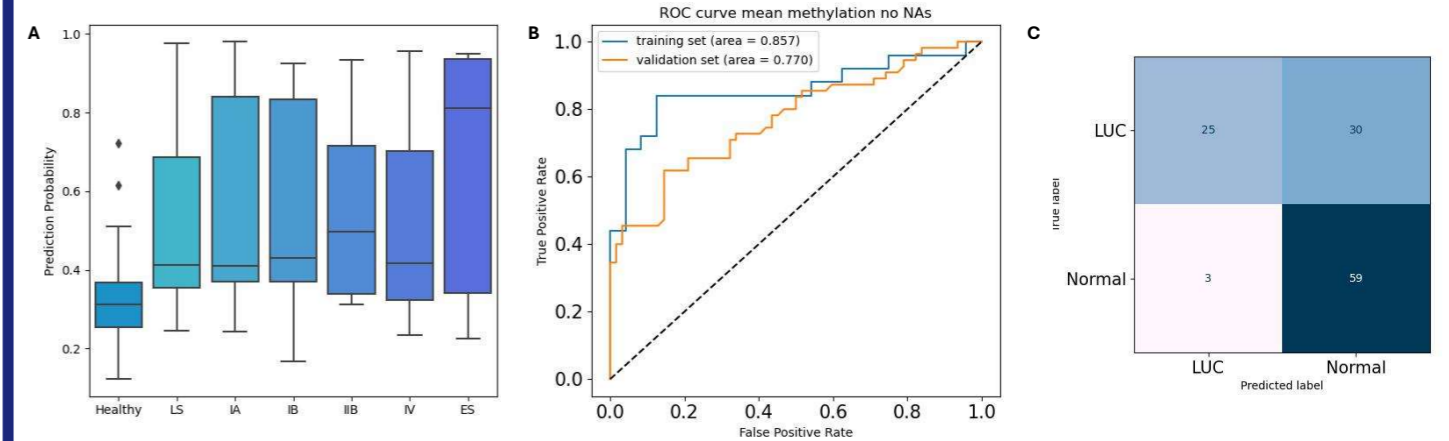


Figure 3. Lung cancer model performance on independent test set. A: Box plot prediction probability per stage. B: ROC curve train/test sets. C: Confusion matrix test set.

## Results:

- DMRs selected: lung cancer 72, breast cancer 74, colorectal cancer 121
- Many tissue derived DMRs show differential methylation in cfDNA
- Models set for high specificity maintain high specificity in independent test cohort

Table 1. Model performance metrics set for high specificity

	LUC - LOOCV	LUC-Test	CRC - LOOCV	BRCA - LOOCV
Accuracy	0.78	0.74	0.83	0.86
Sensitivity	0.59	0.49	0.65	0.72
Specificity	0.96	0.97	1.00	1.00

## Conclusion:

- Tissue derived differentially methylated regions are detectable in cfDNA
- A highly specific model can be made using tissue derived methylation biomarkers
- More research is required to fully leverage years of tissue-based research into the field of liquid biopsies

## References:

- TCGA Research Network: <https://www.cancer.gov/tcga>.
- Ul Haq S, Schmid S, Aparnathi MK, et al. Cell-free DNA methylation-defined prognostic subgroups in small-cell lung cancer identified by leukocyte methylation subtraction. *iScience*. 2022;25(12):105487. Published 2022 Nov 4. doi:10.1016/j.isci.2022.105487
- Shen SY, Singhania R, Fehring G, et al. Sensitive tumour detection and classification using plasma cell-free DNA methylomes. *Nature*. 2018;563(7732):579-583. doi:10.1038/s41586-018-0703-0

## Corresponding author:

Edward@renovarocube.com